

Study Models

Recommended Products

The quality of your study models is determined by the quality of your impressions. In order to receive the highest quality digital study model possible, we recommend you attach **intraoral photos** to your Rx form with every case and use a **wax bite**. Below are the products that we recommend.

Impression Material Options

Using a 5 day alginate is very important. Each of the options below are 5 day alginates that we recommend.

Identic Alginate



www.kerrdental.com

Alginmax



www.pattersondental.com

imprEssix



www.essix.com

Hydrogum 5



www.zhermack.com

Blu-Mousse VPS Bite Registration Material



www.pattersondental.com

Wax Bite / Bite Register Material

We highly recommend Blu-Mousse VPS Impression Material over any other wax bite registration. Wax can bend or break in shipping and is very sensitive to temperature; things that can really distort the wax bite. If you are concerned about proper occlusion, using VPS (or PVS) bite registration material really helps our technicians achieve accurate articulation.

Debubbler

Occasionally, we receive stone models that have holes or bubbles in them. If you pour up your own models, we highly recommend that you use the blue debubbler found on the right. This debubbler helps prevent bubbles from forming when pouring up impressions and will save you time and money.

Blue Debubbler



www.henryschein.com

*Use debubbler when pouring up stone models