
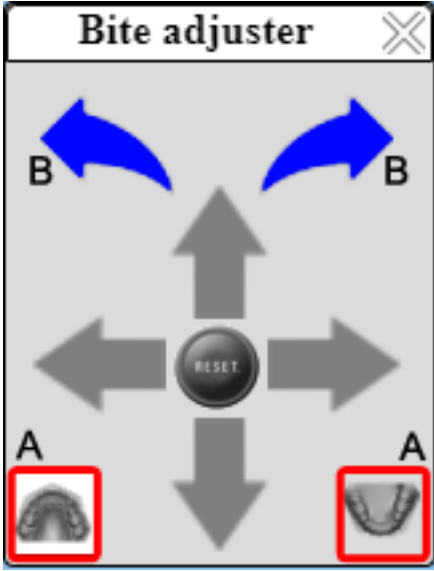
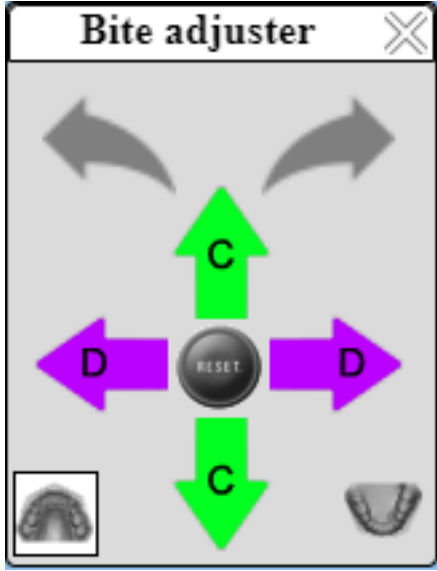


# Bite Adjustment Instructions

OrthoSelect will always adjust any misaligned bites you might have, but in order to save time, this document will serve as a guide to show you how quick and easy you can adjust bites on your own.

<p><b>Step 1:</b> Select the Bite Adjustment tool on the right side of the screen.</p> 	<p><b>Step 2:</b> Use bottom icons (A) to choose which arch to adjust. Use top arrows to rotate the arch (B).</p> 	<p><b>Step 3:</b> Use the four arrows to move the arch up and down (C) or side to side (D).</p> 
---	---	--

**Auto Save:** Any adjustments made are always automatically saved.

**Reset Button:** If you want to reset the bite to the original position, click the “Reset” button found in the middle on the Bite Adjuster window.

**Pro Tip:** To avoid the need to adjust and ensure an accurate bite, always include intra-oral photos on your Rx form when submitting a case. If sending physical impressions, we also recommend using Blu-Mousse Bite Registration Material.

**Note:** If you still would rather us do the adjustments, please send your intraoral photos to [info@myorthoselect.com](mailto:info@myorthoselect.com) along with doctor and patient names.