

# DIBS Bonding

by OrthoSelect

## STEP 1

Apply pumice to clean, then wipe. Apply etch for recommended time, then wipe.



## STEP 2

Apply a thin layer of primer to the center of the prepared teeth.

## STEP 3

UV cure primer by waving curing unit 3x over teeth to make primer tacky.



## STEP 4

Apply adhesive/ composite to the four corners of each bracket.

## STEP 5

Roll on tray from buccal to lingual (front to back) to prevent flash.



## STEP 6

Apply pressure to tray while UV curing each bracket 3x at 4 second intervals.

## STEP 7

Pull off DIBS tray with scaler from mesial to distal.



\*For best results, light cure brackets for a final time before step 7 by waving UV curing unit over each quadrant. \*