



- Log into your account on **syncdibsai.com**
- Click on the **Patients** tab on the left.

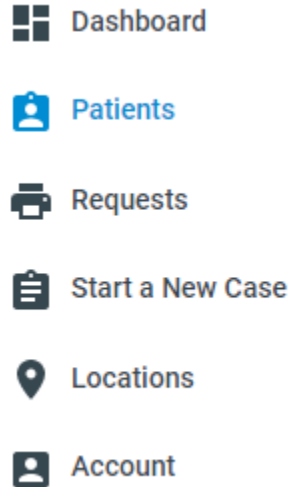


- Search for the patient you'd like to print on this page.
- Then click on the **Details** button to the right under **Actions**.

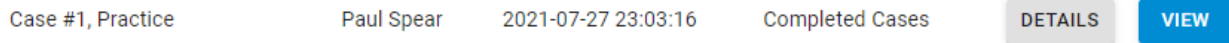

- Scroll down this page until you find the **STL Files** section.

STL Files

Name	Creation Date	Modification Date	Actions
IOS Case #1, Practice IDB Tray Maxillar.stl	2020-12-19 16:06:43	2020-12-19 16:06:44	

- Once there, click on the cloud button for each tray piece to download it.
- Then they will be in your **Downloads** folder, ready to upload to your printer software!